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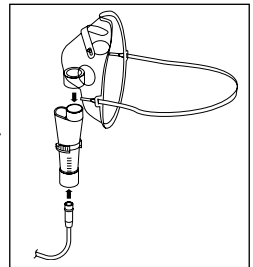
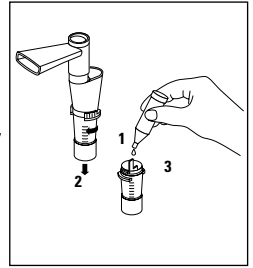
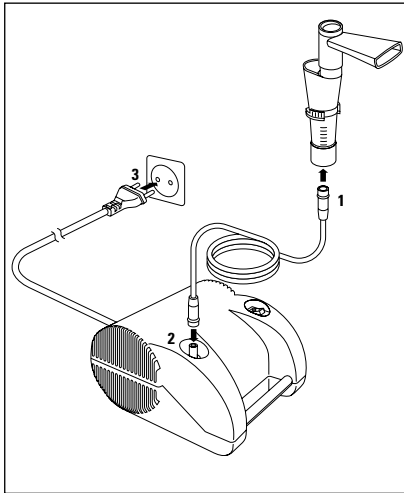
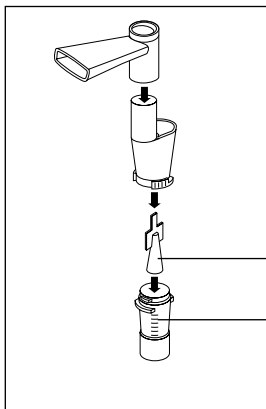
Table of contents

1. Introduction

2. Preparation and usage of the device

3. Cleaning

4. Maintenance, care, and service

5. Malfunctions and actions to take

6. Additional safety instructions

7. Guarantee

8. Technical specifications

Device symbols

 **ON** (power)

Attention: Read the instructions carefully

 **OFF** (power)

Alternating current

 Type B

Class II

Separate collection

Respironics New Jersey, Inc.
5 Wood Hollow Road
Parsippany, NJ 07054 / USA

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1065620 Rev 01

1. Introduction

Your new Microlife nebuliser is a high-quality device for inhalation therapy for asthma, chronic bronchitis, and other respiratory illnesses.

The device is capable of generating a medication mist of particles that are small enough to reach even the deepest regions of your lungs and be of maximal benefit.

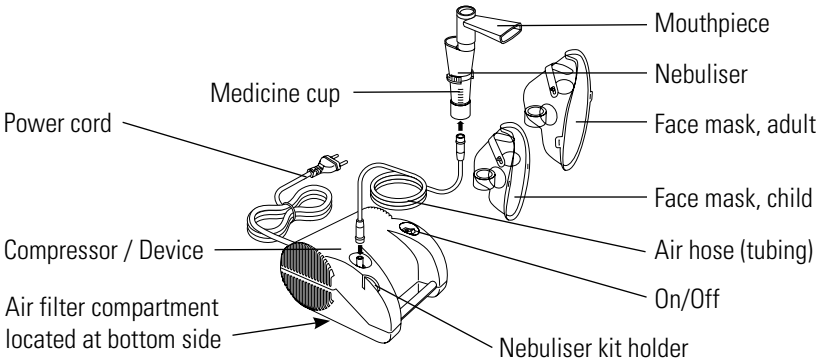
Please only use the medication prescribed to you by your doctor and follow your doctor's instructions with regard to dosage, duration, and frequency of the therapy. For safety reasons, the device and all its accessories should only be used by one person.

The operation and usage of the device is very straightforward. All common liquid medication can be used for inhalation therapy.

Electromagnetic information: Portable and RF communication devices such as cellular phones, pagers, etc. can interrupt operation of electrical medical equipment. For this reason, your compressor must be placed far enough away from these devices to prevent interference. This device meets IEC60601-1-2 standard for electromagnetic compatibility (EMC).

Prior to using the device, please read this manual carefully and retain it in a safe place for future reference.

The figure shows the Microlife inhalation device:

2. Preparation and usage of the device

Prior to using the device for the first time, we recommend cleaning it as per the instructions in this manual.

Please refer to the foldout page with the figures on the correct usage of the device at the beginning of the manual. Do not use the device if you think it is damaged or notice anything unusual.

- 1.** Assemble the nebuliser as shown in the figures. Ensure that all parts are complete.
- 2.** Fill the nebuliser with the inhalation solution as per your doctor's instructions. Ensure that you do not exceed the maximum level.
- 3.** Connect the nebuliser to the device using the air hose and then plug the device into the electricity supply (230V~ / 50Hz).
- 4.** Now switch on the device (I) and place the mouthpiece in the mouth or fit the mask. Make sure mist is visible, coming from the mouthpiece/mask. If possible use the mouthpiece as this gives a better drug delivery to the lungs. However, you may prefer to use the facemask and you will find two sizes are supplied to suit different size faces. The mask should completely enclose the mouth and nose areas. Take the mouthpiece between your teeth, surround it with your lips and breathe in and out slowly through your mouth or place the mask in position over nose and mouth and breathe slowly in and out through your mouth.
- 5.** During the therapy, breathe in and out calmly. Sit in a relaxed position with the upper part of the body upright. Stop inhalation if you feel unwell.
- 6.** After completion of the inhalation period recommended by your doctor, please switch off the device (0) and unplug it from the electricity supply. Disconnect the tubing and clean the nebulizer using the instructions listed on the Cleaning instruction section of the Instructions for use.
- 7.** Empty the remaining medication from the nebuliser and clean the device as per the instructions in this manual.

3. Cleaning

- 1.** Disconnect the nebulizer from the tubing. Unscrew the top from the medicine cup, rinse out the nebulizer and refill with clean water. Reassemble the top and medicine cup, reconnect the tubing and run the nebulizer again for a few seconds to flush out any drug trapped in the nebulizer jets.
- 2.** Take apart the nebulizer.

3. Disconnect the tubing from the medicine cup.
4. Wash all items (except tubing) in hot soapy water.
5. Rinse all parts in clean water. Shake off excess water and allow to air dry. Do not towel dry.
6. Reassemble the nebulizer and insert into compressor device.
7. Turn on the device and let it run for 5 -10 minutes to dry the components. This should eliminate any residue which could obstruct the air and medication flow.

Once a week:

1. Disassemble the nebulizer.
2. Boil the nebulizer and mouthpiece in water with 2 -3 drops of dish washing liquid for 10 minutes.
3. Rinse all parts in clean water for at least 2 minutes.
4. Shake out excess water and allow to air dry.
5. Reassemble and store the nebulizer in a clean place when not in use.

- **Never immerse face masks and the air hose into hot water!**
- **The device must not come into contact with water or heat!**

The face mask should be rinsed out with water after use. The device and the air hose should be wiped out once a day with a clean, moist cloth – **the device must not be plugged into the electricity supply during this process!**

 It is very important to carefully follow the cleaning and disinfecting instructions for efficient and safe product performance.

 **Warning!** Clogged jets or improper assembly may prevent proper delivery of medication.

 **Caution:** To reduce the risk of infection, clean and disinfect the nebulizer between treatments.

Never use a cleaning brush or put sharp objects into the jet hole or baffle as this will damage the nebulizer.

4. Maintenance, care, and service

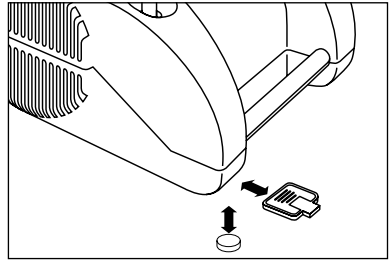
You can procure all necessary spare parts from your dealer or pharmacist, or please contact Microlife directly for details of your local Microlife supplier.

 Use original manufacturer-supplied nebulizers only.

You will find detailed user information on our products and services at www.microlife.com.

We recommend replacing the nebuliser, mouthpiece, and masks after 3 months use.

The filter should be continually checked for cleanliness and replaced if dirty, or after a maximum of 3 months use. Spare filters are provided with the device. To replace the filter, open the compartment on the bottom side of the device and exchange the filter as shown.

 Use original manufacturer-supplied filters only.

 Do not use the device without the filter.

Maintenance and repairs

Never open the device. There are no serviceable parts within the unit. The compressor requires no lubrication or maintenance.

5. Malfunctions and actions to take

Malfunction	Action
The device cannot be switched on.	<ol style="list-style-type: none">1. Check whether the device is correctly plugged into the electricity supply.2. The device has switched off automatically due to overheating. Wait until it has cooled down.
The nebuliser only functions poorly or not at all.	<ol style="list-style-type: none">1. Check whether the air hose is correctly connected at both ends.2. Make sure that the air hose is not kinked or blocked.3. Check whether the nebuliser is correctly assembled.4. Make sure that medication has been added.

The service team at Microlife will be pleased to provide assistance if you have any questions or problems!

6. Additional safety instructions

- If there is visible damage to the device or power cord, please stop using the device.

- The device is only allowed to be used for the purpose described in the instructions. The manufacturer is not liable for damages due to incorrect usage.
- Do not subject the device to extreme temperatures, moisture, dust or direct sunlight.
- Never try to repair the device by yourself. Unauthorised opening of the device will render the guarantee void.
- Ensure that children do not use the device without supervision; some parts are so small that they could be swallowed and present a choking hazard.
- If the device has received a hard knock (fallen on the floor), you find a malfunction or damage, or notice something unusual, you should have the device checked by your national Microlife customer service organisation.
- Do not use in a ventilator circuit.

7. Guarantee

This device is **guaranteed for 2 years** from the date of purchase. The guarantee covers the device, not the replaceable components like the nebuliser, masks, mouthpiece, air hose, and filter. The guarantee is void if the device is improperly used or changes made by third parties. The guarantee is only valid on presentation of the bill or a guarantee card completed by the dealer or pharmacy you purchased it from.

8. Technical specifications

Nebulisation rate:	0.3 ml/min. (3 ml inhalation takes less than 10 minutes)
Particle size:	1.9 µm (MMAD with 0.9% NaCl using API system) or 4.48 µm (MMD with Salbutamol using Malvern system)
Compressor air flow:	9 l/min.
Acoustic noise level:	58 dBA
Power source:	230 V~, 50 Hz
Power cord length:	2 m
Operating temperature:	10 - 40 °C; 10 - 95% relative humidity maximum
Storage temperature:	-25 - +70 °C; 10 - 95% relative humidity maximum
Operating limits:	30 min. ON / 30 min. OFF
Weight:	1560 g
Size:	195 mm x 160 mm x 105 mm
Reference to standards:	EN60601-1, EN60601-1-2 (EMC)

Performance information provided above may vary for drugs supplied in suspension or high viscosity form.

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